

Sabouraud Broth (SABB)

In vitro diagnostic medical aid.

Only for professional use.

Purpose: The product is designed for the selective growth of microscopic fungi.

Package contents: 5 ml in a test tube

Composition: Peptone, yeast extract, L-cysteine hydrochloride, hemin, buffer, glucose, sodium chloride, disodium hydrogen phosphate, demineralized water

Storage: Store in a cool place at +2 to +8 °C.

Usability: Use within 24 hours after opening the primary packaging.

Procedure: Inoculate the clinical material aseptically in the growth medium.

Incubation: under optimal conditions – 24 hours at 35 °C.

Information for users: Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

Instructions in case of a damaged packaging: In case of damage to the packaging, do not use and dispose of the product.

Quality control:

Type of	Inoculated culture	Result
Positive check	<i>Candida albican</i> CCM8215 <i>Aspergillus niger</i> CCM8189	Culture growth
Negative check	Staphylococcus aureus CCM3953	Without a change

Manufacturer

EnviroLab s.r.o.
Galvaniho 7
821 04 Bratislava
Slovak Republic

Place of business:

Obchodná ulica 7
946 51 Nesvady
Slovak Republic