

---

**Schaedler Anaerobic Broth (VLB)**

---

In vitro diagnostic medical aid.

**Only for professional use.**

---

**Purpose:** The product is intended for the growth of facultatively anaerobic and strictly anaerobic bacteria.

**Package contents:** 5 ml in a test tube

**Composition:** Peptone, yeast extract, L-cysteine hydrochloride, hemin, buffer, glucose, sodium chloride, disodium hydrogen phosphate, demineralized water

**Storage:** Store in a cool place at +2 to +8 °C.

**Usability:** Use within 24 hours after opening the primary packaging.

**Procedure:** Inoculate the clinical material aseptically in the growth medium.

Incubation: under optimal conditions – 24 hours at 35 °C.

**Information for users:** Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

**Instructions in case of a damaged packaging:** In case of damage to the packaging, do not use and dispose of the product.

**Quality control:**

Type of	Inoculated culture	Result
Positive check	<i>Clostridium perfringens</i> CCM 4435 <i>Bacteriodes fragilis</i> CCM 4712	culture growth
Negative check	uninoculated medium	without a change

**Manufacturer**

EnviroLab s.r.o.  
Galvaniho 7  
821 04 Bratislava  
Slovak Republic

**Place of business:**

Obchodná ulica 7  
946 51 Nesvady  
Slovak Republic