
Agar for Anaerobic Bacteria with 7% Sheep Blood (AKA)

In vitro diagnostic medical aid.

Only for professional use.

Sterile

Purpose: The product is intended for the cultivation of sporulated and non-sporulated anaerobic microorganisms.

Package contents: 18 ml in a Petri dish

Composition: Peptone, L-cysteine, L-arginine, sodium chloride, glucose, starch, vitamin complex, buffer, agar, demineralized water

Storage: Store in a cool place at +4 to +6 °C.

Usability: Use within 24 hours after opening the primary packaging.

Procedure: Inoculate the clinical material aseptically on the medium surface.

Incubation: under optimal conditions 18 – 24 hours at 35 °C.

Information for users: Do not use the medium if it shows signs of microbial contamination, color change, drying, cracking or other signs of damage. All specimens and inoculated products must be considered as potentially infectious and safety measures must be followed in accordance with applicable regulations when handling them.

Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

Instructions in case of a damaged packaging: In case of damage to the packaging, do not use and dispose of the product.

Quality control:

Type of inspection	Inoculated culture	Result
positive check	Clostridium perfringens CCM 4435 Bacteroides fragilis CCM 4712	growth
negative check	Escherichia coli CCM3954	without a change

Manufacturer

EnviroLab s.r.o.
Galvaniho 7
821 04 Bratislava
Slovak Republic

Place of business:

Obchodná ulica 7
946 51 Nesvady
Slovak Republic