

---

**Blood Agar No. 4 with 7% Horse Blood – Chocolate (CAK)**

---

In vitro diagnostic medical aid.

**Only for professional use.**

**Sterile**

---

<b>Purpose:</b>	The product is intended for the cultivation and isolation of growth-intensive bacteria.
<b>Package contents:</b>	18 ml in a Petri dish
<b>Composition:</b>	Peptone, starch, sodium chloride, agar, horse blood, demineralized water
<b>Storage:</b>	Store in a cool place at +4 to +6 °C.
<b>Usability:</b>	Use within 24 hours after opening the primary packaging.

**Procedure:** Inoculate the clinical material aseptically on the medium surface.

Incubation: under optimal conditions – 24 hours at 35 °C.

**Information for users:** Do not use the medium if it shows signs of microbial contamination, color change, drying, cracking or other signs of damage. All specimens and inoculated products must be considered as potentially infectious and safety measures must be followed in accordance with applicable regulations when handling them.

Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

**Instructions in case of a damaged packaging:**

In case of damage to the packaging, do not use and dispose of the product.

**Quality control:**

Type of inspection	Inoculated culture	Result
positive check	Streptococcus pneumoniae CCM4424 Streptococcus pyogenes CCM4425	culture growth
negative check	uninoculated medium	without a change

**Manufacturer**

EnviroLab s.r.o.  
Galvaniho 7  
821 04 Bratislava  
Slovak Republic

**Place of business:**

Obchodná ulica 7  
946 51 Nesvady  
Slovak Republic