

Bile Esculin Azide Agar (ESC2)

In vitro diagnostic medical aid.

Only for professional use.

Sterile

Purpose: The product is used for the isolation and preliminary identification of group D enterococci and streptococci.

Package contents: 2 ml in a test tube

Composition: Peptone, bovine bile, casein peptone, esculin, ferric citrate, sodium chloride, yeast extract, sodium azide, agar, demineralized water

Storage: Store in a cool place at +4 to +6 °C.

Usability: Use within 24 hours after opening the primary packaging.

Procedure: Inoculate the clinical material aseptically on the medium surface.

Incubation: under optimal conditions – 24 hours at 35 °C.

Information for users: Do not use the medium if it shows signs of microbial contamination, color change, drying, cracking or other signs of damage. All specimens and inoculated products must be considered as potentially infectious and safety measures must be followed in accordance with applicable regulations when handling them.

Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

Instructions in case of a damaged packaging: In case of damage to the packaging, do not use and dispose of the product.

Quality control:

Type of inspection	Inoculated culture	Result
positive check	Enterococcus faecalis ATCC19433 /CCM4224	growth
negative check	Streptococcus pyogenes ATCC12344/CCM4425	inhibited growth

Manufacturer

EnviroLab s.r.o.
Galvaniho 7
821 04 Bratislava
Slovak Republic

Place of business:

Obchodná ulica 7
946 51 Nesvady
Slovak Republic