
Listeria sp. Selective Agar (LIA)

In vitro diagnostic medical aid.

Only for professional use.

Sterile

Purpose: The product is designed for the cultivation and isolation of *Listeria monocytogenes*.

Package contents: 18 ml in a Petri dish

Composition: Enzymatic casein hydrolyzate, peptone, dextrose, sodium chloride, thiaminium dichloride, tryptaflavin, nalidixic acid, agar, potassium thiocyanate, colistin, moxalactam, demineralized water

Storage: Store in a cool place at +4 to +6 °C.

Usability: Use within 24 hours after opening the primary packaging.

Procedure: Inoculate the clinical material aseptically on the medium surface.

Incubation: under optimal conditions – 24 hours at 35 °C.

Information for users: Do not use the medium if it shows signs of microbial contamination, color change, drying, cracking or other signs of damage. All specimens and inoculated products must be considered as potentially infectious and safety measures must be followed in accordance with applicable regulations when handling them.

Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

Instructions in case of a damaged packaging: In case of damage to the packaging, do not use and dispose of the product.

Quality control:

Type of inspection	Inoculated culture	Result
positive check	<i>Listeria monocytogenes</i> ATCC19117/CCM4699	abundant growth of culture
negative check	<i>Staphylococcus aureus</i> CCM3953	poor or no growth

Manufacturer

EnviroLab s.r.o.
Galvaniho 7
821 04 Bratislava
Slovak Republic

Place of business:

Obchodná ulica 7
946 51 Nesvady
Slovak Republic