
Sabouraud Broth (SABB)

In vitro diagnostic medical aid.

Only for professional use.

Sterile

Purpose: The product is designed for the selective growth of microscopic fungi.

Package contents: 500 ml solution in a bottle

Composition: peptone (10 g/l)
glucose (20 g/l)
casein peptone (5 g/l)
demineralized water

Storage: Store in a cool place at +2 to +8°C.

Usability: Use within 24 hours after opening the primary packaging.

Procedure: Inoculate the sample in the SABB culture medium and incubate at 25 °C for 48 hours.
After the incubation period, inoculate onto solid culture media to capture yeast and mold.

Information for users: Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

Instructions in case of a damaged packaging: In case of damage to the packaging, do not use and dispose of the product.

Quality control:

Type of inspection	Inoculated culture	Result
Positive check	<i>Candida albicans</i> CCM8215 <i>Aspergillus niger</i> CCM8189	Culture growth
Negative check	<i>Staphylococcus aureus</i> CCM3953	Without a change

Manufacturer

EnviroLab s.r.o.
Galvaniho 7
821 04 Bratislava
Slovak Republic

Place of business:

Obchodná ulica 7
946 51 Nesvady
Slovak Republic