

Thioglycollate Broth (TG)

In vitro diagnostic medical aid.

Only for professional use.

Purpose: The product is designed for the non-selective growth of anaerobic and aerobic bacteria – the culture medium prescribed by PhEur for sterility tests.

Package contents: 4 ml, 5 ml and 10 ml of the solution in a test tube, 50 ml in the NTS bottle

Composition: Casein peptone, yeast extract, glucose, L-cystine, resazurin, sodium thioglycolane, sodium chloride, agar, demineralized water

Storage: Store in a cool place at +2 to + 8°C.

Usability: Use within 24 hours after opening the primary packaging.

Procedure: Inoculate the clinical material aseptically in the growth medium.

Incubation: under optimal conditions – 24 hours at 35 °C.

Information for users: Do not use after the expiration date.

Dispose of used and unused products as hazardous waste in accordance with applicable regulations.

Instructions in case of a damaged packaging: In case of damage to the packaging, do not use and dispose of the product.

Quality control:

Type of inspection	Inoculated culture	Result
Positive inspection	<i>Staphylococcus aureus</i> CCM4516 <i>Pseudomonas aeruginosa</i> CCM1961 <i>Clostridium sporogenes</i> CCM4409	Growth in a form of turbidity
Negative check	uninoculated medium	Without a change

Manufacturer

EnviroLab s.r.o.
Galvaniho 7
821 04 Bratislava
Slovak Republic

Place of business:

Obchodná ulica 7
946 51 Nesvady
Slovak Republic